

# Surface Check Testing Kit

## Detailed Instructions for Use



### PREPARATION

- Fill out the **Surface Information Card** (Please return this card with the samples.):
  - Each number corresponds to a collection tube in the kit.
  - We recommend the following surfaces for testing:  
Door knobs, TV remote, faucet handles, bedside tables, light switches, ventilation exits, soap/sanitizer dispensers, handrails, phones, or toilets.  
  
Multiple surfaces can be tested or multiple points on the same surface can be tested.
- Put on a pair of gloves.
- You are now ready to collect samples.

### For each surface to be sampled:

- Remove a collection tube from the Sample Collection Kit.
- Open the collection tube and set it upright near the surface that will be sampled. Place the cap near the collection tube.
  - The collection tube has 1 ml of fluid inside of it, be careful not to spill this fluid when handling the open tube.
    - Any spilled fluid can be cleaned with typical cleaning products.
    - Any contact with skin should be cleaned with soap and water.
  - The tamper-evident sealing ring will detach from the cap when it is opened. This can be discarded.
- Collect the sample from the surface:
  - Remove a swab from the swab pouch.
  - Slide the swab across the sample area (using light pressure). Be sure to use both flat sides of the fabric swab head.
  - Hold the tube firmly with one hand and insert the swab head into the tube. Cover the mouth of the tube with your thumb and break off the swab head at the notch.
  - The swab head will fall into the collection tube.
  - Close the collection tube and discard the swab handle.
  - Place the collection tube into the Surface Collection Kit pouch.
  - Repeat for each surface as needed.

### COLLECTION

- Seal the Sample Collection Kit and place it into the Box **with the Surface Information Card.**
- Close the box.

**If the box was shipped to you:**

- A pre-labeled UN3373 overpack was provided, simply insert your box into this overpack and seal it.
- Request a pickup via [www.fedex.com](http://www.fedex.com).

**If the box was delivered via an Enviral Tech courier:**

- Contact us at (541) 362-1361 to schedule a pickup.
- Results will be communicated via email within 24 hours from receipt at our facility.

Please read all of the instructions on this sheet prior to using this kit.

**⚠ THIS TEST KIT IS NOT INTENDED FOR HUMAN DIAGNOSTIC PURPOSES**

Support for this kit is available 7 days a week.

(541) 362-1361 | [support@enviraltech.com](mailto:support@enviraltech.com)

**Contents list:**

- Sample Collection Kit
- Detailed Instruction Sheet
- Surface Information Card
- Large Disinfecting Wipe
- Pair of Disposable Nitrile Gloves